

PRX-T[®]₃₃

NO-NEEDLES BIOREVITALISATION

MEDICAL DEVICE **CE**



wiQO[®]med

PRX-T₃₃[®]

It is a Class I medical device with a **patented formula** that regulates hydrogen peroxide (H₂O₂) and trichloroacetic acid (TCA) in active concentrations. It is a **non-injectable biorevitaliser** that works **without damaging the epidermis**. It can be used **alone** or **in combination with common aesthetic medicine treatments** to improve its effectiveness.


Innovation

PRX-T₃₃[®] has a **special innovative formula** that combines the positive effects of these three components into one product:

HYDROGEN PEROXIDE (H₂O₂)

33% TRICHLOROACETIC ACID (TCA)

5% KOJIC ACID

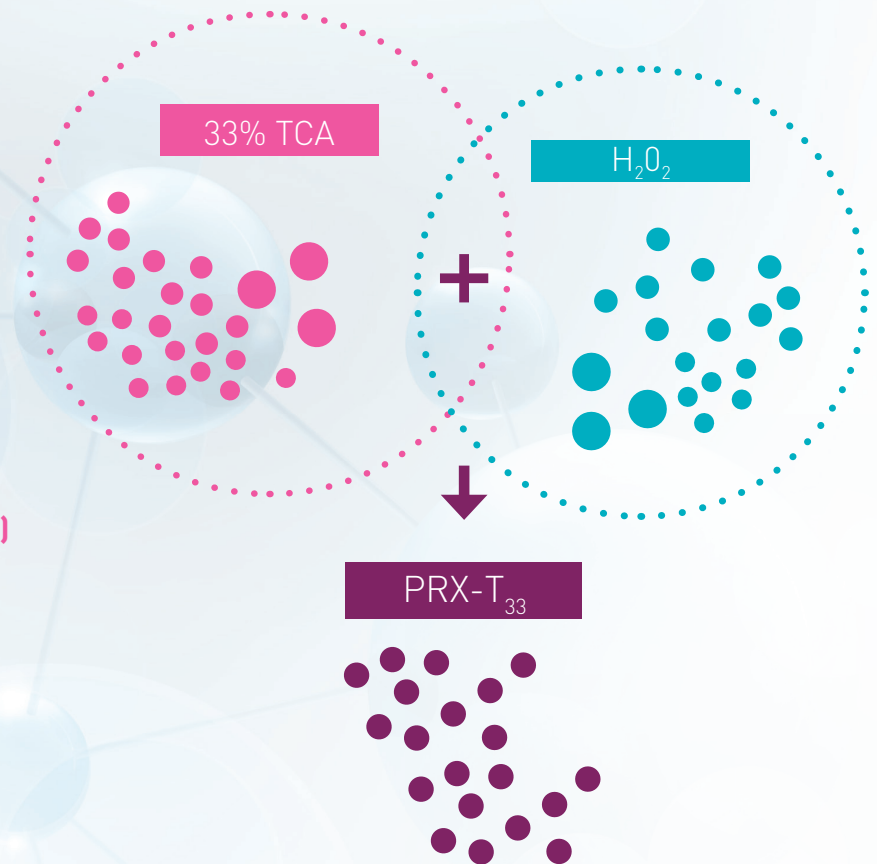


H₂O₂ Is a substance that occurs naturally in cells, which are able to synthesize it.

Scientific studies have shown that LOW CONCENTRATIONS of H₂O₂:

- promote the **healing of wounds**
- help **remodel the extracellular matrix**
- **regulate inflammation**

TCA Is a chemical agent that has long been used for peeling and treating damaged skin.


Kojic acid Is an effective skin-whitener and depigmenting agent. It plays an important role in inhibiting the post-inflammatory hyperpigmentation that can occur in susceptible individuals.

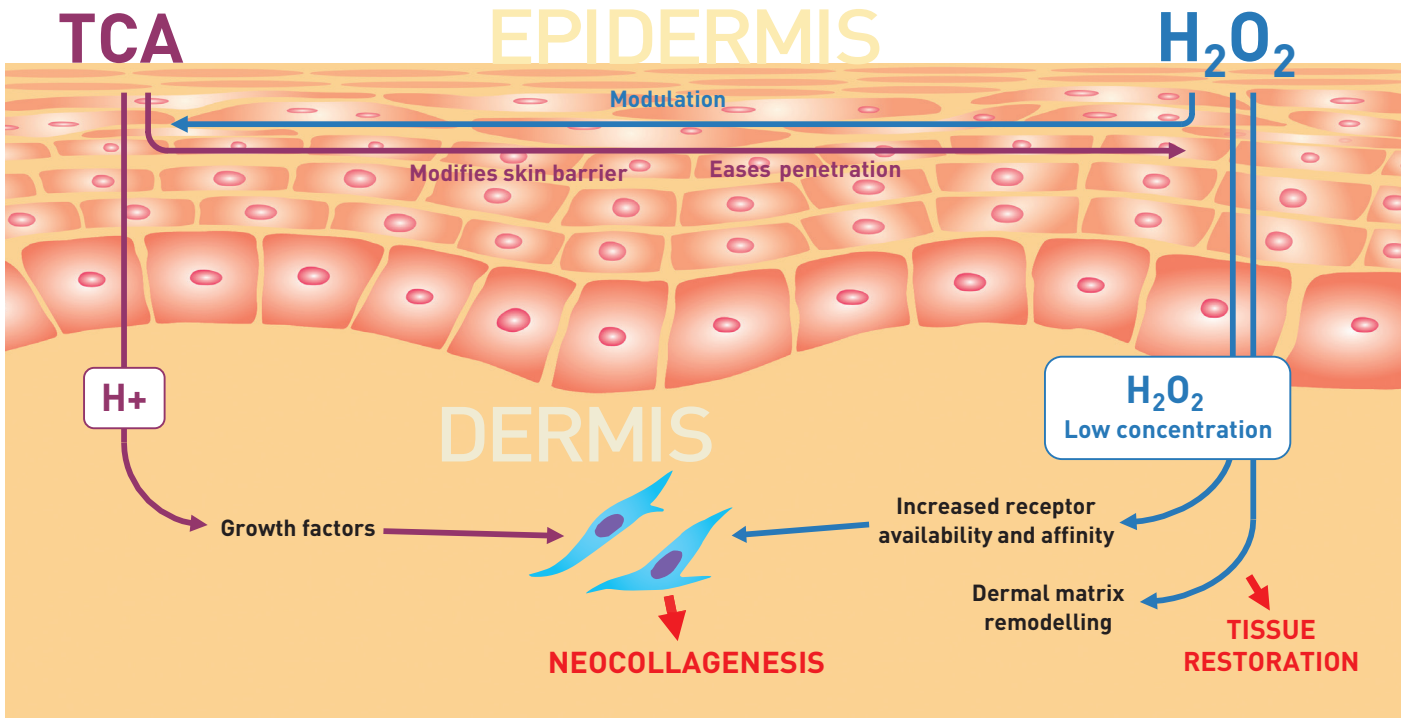

Scientific studies have shown that applying **TCA** to the skin leads to an increase in a pro-hormone called **Pro-opiomelanocortin¹ (POMC)** on the skin. At an **acidic pH**, **POMC** is cut by enzymes at the level of the **Golgi apparatus²** creating different

types of molecules which can lead to: an intense dermal stimulation, proliferation of keratinocytes and fibroblasts, an increase in cell turnover, better barrier effect, neocollagenogenesis.

Golgi apparatus

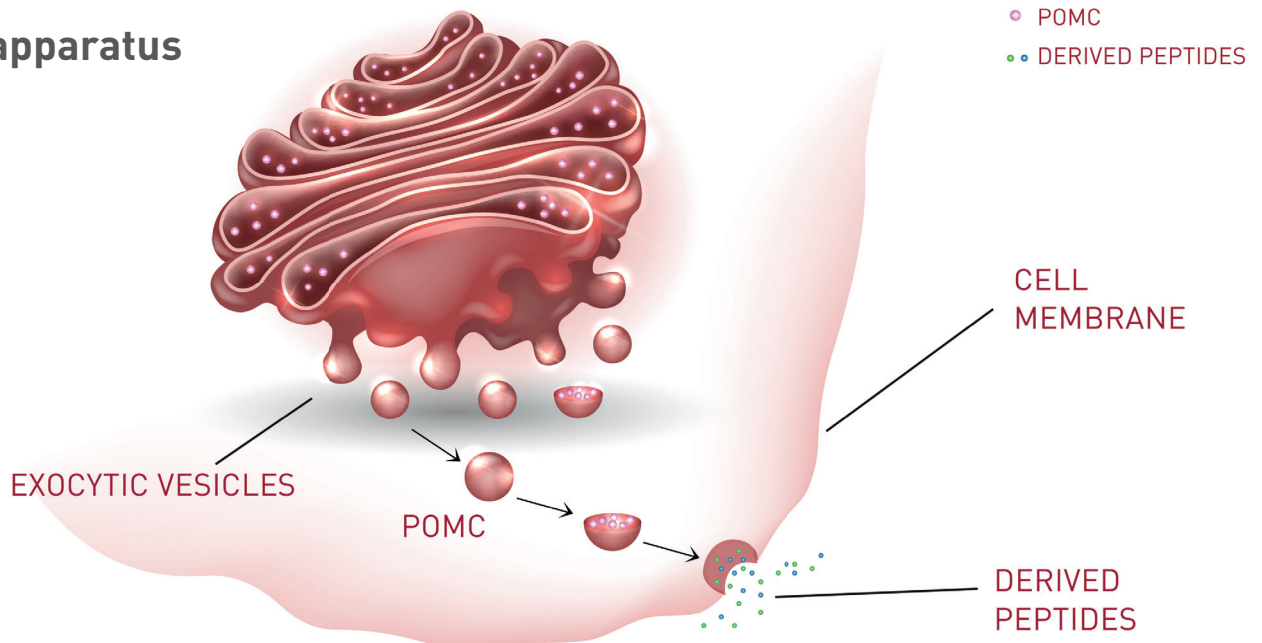

1 Influence of chemical peeling on the skin stress response system. Kimura A, Kanazawa N, Li HJ, Yonei N, Yamamoto Y, Furukawa F. *Exp Dermatol.* 2012 Jul;21 Suppl 1:8-10. doi: 10.1111/j.1600-0625.2012.01495

2 Proteolytic processing of pro-opiomelanocortin occurs in acidifying secretory granules of AtT-20 cells. Tanaka S1, Yora T, Nakayama K, Inoue K, Kurosumi K. *J Histochem Cytochem.* 1997 Mar;45(3):425-36.]

Indications

Easy for the doctor, **pleasant** for the patient

- **no needles**
- **does not cause pain**
- **no interruption** in social life
- **non-photosensitizing** and therefore can even be done in the summer
- duration of the treatment is **10-15 minutes**
- produces **immediate and evident firming**

Suitable for **both sexes at any age** and **during any season** because **it is non-photosensitizing**

5-12 years • scars (chickenpox, impetigo)

12-25 years • replaces facial cleansing (smooths the skin and aids in the removal of blackheads and pustules)
• stretch marks

25-35 years • melasma
• stretch marks

35-45 years • biorevitalization of the face
• biorevitalization of the body
• prevention of ageing of the skin
• relaxation of the breast skin

> 45 years • treatment of hypotonic skin
• relaxation of the breast skin

Basic treatment application method

1.

Clean the area of the skin to be treated with a mild soap

2.

Dry well



3.

Apply the **PRX-T₃₃** to the skin using the applicator brush or by rubbing

4.

2-5 applications, depending on the thickness of the skin, until a visible and/or evident turgor is obtained

5.

Rinse with running water

6.

Apply a specific nourishing and moisturizing cream to restore the skin's hydro-lipid film removed by the **PRX-T₃₃**



Go to this link
<https://youtu.be/Rockoqlv3wA>
and watch the PRX-T₃₃ APPLICATION video

Combinations

PRX-T₃₃ is effective as a single treatment but can also be used **in combination** with other aesthetic medicine treatments to **enhance the effects**.

It can be combined with:

- micro-needling
- fillers
- injection biorevitalization
- fractional non-ablative lasers
- mechanical skin stimulation
- platelet rich plasma (PRP)
- photodynamic therapy (PDT)
- polydioxanone threads (PDO)
- carboxy therapy (CDT)
- fractional lasers
- radio frequency (RF)

Visit www.wiqomed.com to download the instructions for use and possible combinations.

PRX-T₃₃
ASSOCIATED WITH INJECTIVE BIOREVITALISATION

PRX-T₃₃ MEDICAL DEVICE

5 x 4 mL

BEFORE APPLICATIONS

AFTER APPLICATIONS
Right eye: Combined treatments, Left eye: injectable bio-revitalisation only

Please contact [wiQO](http://www.wiqomed.com) – scientific division for scientific or outpatient practice queries scientificdiv.gpg@wiqo.com

PRX-T₃₃
ASSOCIATED WITH MICRONEEDLING

PRX-T₃₃ MEDICAL DEVICE

5 x 4 mL

BEFORE APPLICATIONS

AFTER 4 APPLICATIONS

Please contact [wiQO](http://www.wiqomed.com) – scientific division for scientific or outpatient practice queries scientificdiv.gpg@wiqo.com

PRX-T₃₃
PROTOCOL FOR STRETCH MARKS

PRX-T₃₃ MEDICAL DEVICE

5 x 4 mL

BEFORE APPLICATIONS

AFTER 4 APPLICATIONS

Please contact [wiQO](http://www.wiqomed.com) – scientific division for scientific or outpatient practice queries scientificdiv.gpg@wiqo.com

PRX-T₃₃
ASSOCIATED WITH PDO THREADS

PRX-T₃₃ MEDICAL DEVICE

5 x 4 mL

BEFORE APPLICATIONS

AFTER APPLICATIONS

Please contact [wiQO](http://www.wiqomed.com) – scientific division for scientific or outpatient practice queries scientificdiv.gpg@wiqo.com

PRX-T₃₃
ASSOCIATED WITH PLATELET RICH PLASMA (PRP) IN THE TREATMENT OF ALOPECIA

PRX-T₃₃ MEDICAL DEVICE

5 x 4 mL

♂ 22 a.
Stage III to I according to the Hamilton-Norwood scale

♀ 47 a.
Stage I-2 to I-2 according to the Ludwig scale

Please contact [wiQO](http://www.wiqomed.com) – scientific division for scientific or outpatient practice queries scientificdiv.gpg@wiqo.com

Home treatment with the WiQO® Dermocosmetic line

The **health** and **well-being** of the skin is the primary objective of the cosmetics in the **WiQO® ACTIVE SKIN PROGRAM** line. Developed using the most advanced scientific research, they offer **quality** and **effectiveness**. The health of the skin is maintained through a program that satisfies the following main needs: **Protection, Prevention, Stimulation**



SMOOTHING FACE CREAM

All skin types
30 ml

It reduces the depth of wrinkles in mature skin and smooths out imperfections in younger skin: it's the dermocosmetic that is ideal for all skin types, even for men. Our skin has a pH of about 5.5. By applying the **WiQo Smoothing Face Cream** the pH is lowered, causing the skin to do some biological "exercise" to restore it to a normal pH by activating dermal stimulation processes. The skin never gets used to this particular process, so it can be and should be used every day of the year.

55% of subjects had a reduction in wrinkle depth*
100% of subjects had an improvement in skin hydration*

* Test performed on subjects between the ages of 28 and 82 for 4 weeks



NOURISHING AND MOISTURIZING FACE CREAM

Normal or combination skin
50 ml

Proper hydration for normal or combination skin with a perfect mixture of nutrients and protective substances, in one daily application.

+18.7% increase in the average measured hydration level*
86.7% of people were satisfied with the treatment*

* Test performed on subjects between the ages of 30 and 70 for 2 weeks



NOURISHING AND MOISTURIZING FACE CREAM

Dry skin
50 ml

POST FACIAL TREATMENT

A concentrate of nutrients and protective substances that are essential for dry skin. The most valuable ally for a well-nourished and hydrated skin.

+15% increase in the average measured hydration level*
86.7% of people were satisfied with the treatment*

* Test performed on subjects between the ages of 30 and 70 for 2 weeks



HYDRATING AND ELASTICIZING BODY CREAM

200 ml

The properties of both the Smoothing Cream and the Nourishing and Moisturizing Face Cream are combined in this **WiQo Body Cream**, making it suitable for body skin of all ages. The combination of glycolic acid with emollient oils and plant extracts quickly resolves dryness problems, making the treated areas firmer and more elastic. The **WiQo Body Cream** is also indicated as an adjuvant treatment in preventing stretch marks.

100% of subjects had an improvement in skin hydration*
85% of subjects had an improvement in skin elasticity*

* Test performed on subjects between the ages of 28 and 82 for 4 weeks

THE PERFECT EXERCISE FOR YOUR SKIN

WiQO[®]med



THE ASSURANCE OF OVER 300,000 TREATMENTS PERFORMED IN MORE THAN 10 COUNTRIES AROUND THE WORLD

WiQO[®]



WiQO[®]med
Scientific Division

Direct contact with dermatologists and research biologists **WiQO[®]med** for scientific questions or outpatient practice at **PRX-T₃₃**
scientificdiv.gpq@wiqo.com



GPQ srl
Milan - ITALY

Operational Office: Production Laboratory and R&D
Ph.: +39 040 360408 Fax: +39 040 761393 contact@wiqomed.com
Via Flavia, 23/1 - 34148 Trieste - ITALY